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EN

Dear customer

Thank you for choosing a quality product from Dentaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. Should you have any questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. General description

Compact size combined with brilliant characteristics. This describes the premium metal bracket discovery® smart from Dentaurum. It is a consequent development of the discovery® brackets which have been successful on the market for a long time. It complements the bracket range with compact dimensions and a new, innovative slot shape.

3. Intended use

For the orthodontic treatment of malocclusions.

4. Processing

4.1 Informing the patient

Inform the patient about the possible consequences and risks of bracket treatment, in particular during debonding. This applies especially in the case of teeth with enamel that already shows signs of initial damage (cracks, crazing, splitting, etc.).

4.2 Conditioning teeth

Check each tooth thoroughly for any existing damage to the enamel to avoid damaging the tooth during debonding. Follow the instructions from the bracket adhesive manufacturer for conditioning the teeth.

4.3 Selection of bracket adhesives

discovery® smart brackets can be bonded using all commercially available bracket adhesives. We recommend CONTEC lc (REF 163-111-00), ConTec Go! (REF 163-321-00) and SmartBond® (REF 163-506-00).

4.4 Removing and handling

Remove the bracket blister from the packaging. The brackets are protected by two separate foils. Remove the outer protective foil (Fig. 1).

To remove the bracket from its blister packaging, grasp the bracket through the perforation in the foil above each bracket compartment (Fig. 2 a + 2 b) using clamping tweezers (e.g. REF 025-277-00) mesially-distally. The patented laser-structured base of the discovery® smart brackets should not be touched with your fingers or contaminated in any way, as this could otherwise detrimentally affect the bond.

4.5 Applying bracket adhesive

Prepare the bracket adhesive according to the manufacturer’s instructions. Apply a small amount of adhesive to the center of the bracket base, enough to wet the entire bracket base. This guarantees an optimum bond.

4.6 Indirect bonding technique

discovery® smart brackets are bonded on the plaster or the plastic model with the bracket adhesive. The adhesive layer may not be removed afterwards. Indirect bonding techniques using a sugar solution or other water-soluble laboratory adhesives that later have to be completely removed are not suitable. This would destroy the mechanical retention of discovery® smart and a reliable bracket fit could no longer be guaranteed.

Thermoformed splints or silicone transfers, e.g. LinguSil (REF 164-000-00) can be used as a transfer matrix.

4.7 Placing and aligning brackets

After applying the adhesive, place the discovery® smart bracket in the usual way onto the tooth. Place the discovery® smart bracket in the best possible position using the angulation aid on the crimping tweezers. Press the bracket onto the tooth so that excess adhesive is pushed out to the sides. Avoid contact with the antagonist due to the risk of abrasion and/or premature bracket loss. If necessary, a palatal bite raise can be used e.g. Bite-Bumper™ (REF 705-000-00). Completely remove any excess adhesive before it has cured. This is particularly important to ensure easy debonding at the end of treatment.

4.8 Ligating archwires

Once the adhesive has cured (observe adhesive hardening times in the manufacturer’s instructions for use), ligate as usual with wire or plastic ligatures.

4.9 Bracket debonding

The excellent bonding properties of discovery® smart brackets require special attention during debonding:

For the debonding of discovery® smart brackets, we recommend using Weingart pliers (e.g. REF 003-120-00). Grasp the bracket mesially-distally / diagonally by the wings, press the pliers together gently and disengage the base of the bracket carefully from the tooth using the debonding channel. Turn the bracket briefly to the right or the left, without pulling, to remove it from the tooth.

5. Composition

The composition is included in the materials list, refer to the catalog or www.dentaurum.com.

6. Safety instructions

There are no known risks or side effects (allergic reaction) for discovery® smart brackets. The product should not be used if there is a known allergic reaction to one or more of the material components.

7. Storage and shelf life

Store discovery® smart brackets in closed packaging.

8. Information for single use products

discovery® smart brackets are intended for single use only. Due to the mechanical stress during treatment and debonding, neither the dimensions nor the bonding strength can be guaranteed should the brackets be recycled.

Reconditioning of discovery® smart brackets that have already been used (recycling) and their reuse on a patient is not permitted.

9. Additional information

- Technical information such as torque, angulation, size etc. can be found on the relevant product label or in the product catalog.
- There are no particular precautions to be taken when disposing of discovery® smart brackets after treatment.
- Should you have any questions, please contact your local representative.

10. Availability

For the complete range of discovery® smart brackets, please see our catalog or www.dentaurum.com.

11. Quality information

Dentaurum ensures faultless quality of its products. These recommendations are based upon Dentaurum’s own experiences. The user is responsible for the processing of the products. Responsibility for failures cannot be taken by Dentaurum as we have no influence on the processing on site.

12. Explanation of symbols used on the label

 Refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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