DMG MiniDam

Gingival guard, silicone

DMG Chemisch-Pharmazeutische Fabrik GmbH
Elbgastrasse 248 - 22547 Hamburg Germany
www.dmg-dental.com
091956/#3_2015-10

DE	Gebrauchsinformation
EN	Instructions for use
FR	Mode d'emploi
IT	Istruzioni per l'uso
ES	Instrucciones de empleo
PT	Instruções de uso
NL	Gebruiksaanwijzing

Figures

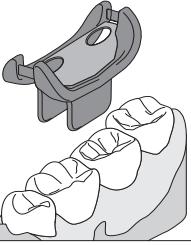
Fig. 1

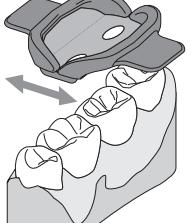
Fig. 2

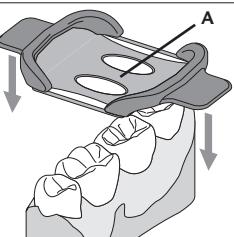
Fig. 3

Fig. 4

Fig. 5

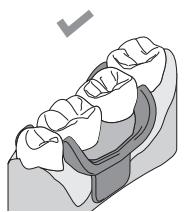
Fig. 6

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Deutsch

Produktbeschreibung

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English

Product description

The gingival guard is comprised of silicone and is used to facilitate a relatively dry environment in the proximal area between two teeth and to protect the gingiva against dental materials.

Indications

- Establishing a relatively dry environment in the case of complete dentition, for example when using Icon or Composite filling materials.

Contraindications

- Not suitable for establishing absolutely dry environments.
- Do not use in cases of allergies to one of the ingredients or in the event of contact allergies.

Basic safety information

- For dental use only.
- Keep out of the reach of children.

Side effects

No systemic side effects are known to date. Hypersensitivity to components of the material or contact allergies cannot be excluded in individual cases.

Instructions for use

Avoid damaging the gingival guard with dental floss or other materials. Do not use damaged gingival guard.

Recommended use

1. Clean the working area.
2. If the teeth are very close together: separate the teeth using a wedge.
3. Pre-stretch the gingival guard by gently pulling the tabs apart.
4. Position the gingival guard over the affected teeth. Make sure that the tabs are pointing in the direction of the area being worked on (Fig. 1).
5. Place the gingival guard over the affected teeth and pull the tabs apart (Fig. 2).
6. Place the interdental section of the gingival guard in the proximal position.

Note: The middle interdental section is used to protect the working area (Fig. 3:A).

7. Pull the gingival guard towards the gingiva until the interdental sections are positioned in the proximal area (Fig. 3).
8. Release the gingival guard (Fig. 4).
9. Fold the gingival guard cervically where the teeth are isolated.
10. Fold the tabs cervically (Fig. 5).
11. Remove wedges as appropriate.
12. Verify that the gingival guard is properly seated and correct if required (Fig. 6).
13. Further measures to facilitate a dry environment (e.g. cotton rolls, aspiration) can be carried out as required.
14. Perform treatment.
15. Remove the gingival guard and discard.

Disposal

After use, dispose of the product in household waste.

Storage and shelf life

- Store in a dry place (15 – 25°C/59 – 77°F) at room temperature.
- Do not use after the expiration date!

Composition

Silicone elastomers, pigments, additives

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